# Askina® Calgitrol® Ag

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## INSTRUCTIONS FOR USE

**1. PRODUCT DESCRIPTION**  
 Askina® Calgitrol® Ag is a sterile dressing consisting of two layers:  
 • an absorbent polyurethane foam layer which provides for the absorption of exudate  
 • an ionic silver alginate matrix which provides for broad antimicrobial effectiveness and helps prevent contamination from external bacteria; in the presence of wound exudate, the silver alginate matrix helps maintain a moist wound environment conducive to natural healing conditions.

**2. INDICATIONS**  
 Askina® Calgitrol® Ag is indicated for the management of excised partial to full thickness wounds, stage I-IV pressure sores, venous ulcers, second degree burns and donor sites. It is also used for external use only and may be used in the management of infected wounds under medical supervision at the discretion of the physician.

**3. PRECAUTIONS**  
 In the event of clinical infection, topical silver does not replace the need for systemic therapy or other systemic antibiotics.

**4. CONTRAINDICATIONS**  
 Askina® Calgitrol® Ag should not be used on:  
 • Patients with a known sensitivity to silver  
 • Askina® Calgitrol® Ag should not be used:  
 • Where the presence of the metal is contraindicated.  
 Askina® Calgitrol® Ag is contraindicated for Ulcers resulting from certain infections or medical conditions, such as:  
 • Tuberculosis,  
 • Deep fungal infection,  
 • Third degree burns.

**5. INFORMATION FOR USE**  
 Askina® Calgitrol® Ag is very simple to use and requires no special skills or equipment. Askina® Calgitrol® Ag may be left in place for up to seven days or until the dressing is saturated with wound exudate.

**5.1 Skin Preparation**  
 a) Cleanse the wound with sterile saline (0.9%) or an appropriate wound cleansing solution such as Protoson® antiseptic solution.  
 b) Dry the skin surrounding the wound.

**5.2 Dressing Application**  
 Select the appropriate Askina® Calgitrol® Ag size that will completely cover the wound surface, ensuring a 3cm margin beyond the edges of the wound. If necessary, several dressings can be overlapped to cover very large wounds. Place the dressing with the silver matrix (dark surface) facing the wound. Secure Askina® Calgitrol® Ag with a secondary dressing or secure with surgical tape or bandage as appropriate.

**5.3 Dressing Removal / Changes**  
 Remove retention dressing if applicable. Gently lift Askina® Calgitrol® Ag from the wound. If the dressing is saturated, more gently irrigate with saline or Ringer solution. Follow procedures 5.1. and 5.2. to apply a new dressing.

**5.4 SPECIAL NOTES**  
 The wound should initially appear to increase in size as the early stages of Askina® Calgitrol® Ag treatment. This is normal and occurs as any wound debris is removed from the edges of the wound. This clears the way for healing. In the management of moderately exuding wounds, Askina® Calgitrol® Ag can only make a difference if the wound is more conducive to healing. There are cases where healing is impaired as a result of underlying conditions. In these instances, Askina® Calgitrol® Ag alone will not improve the wound. A suitable treatment of the underlying conditions will be necessary as well. Therefore, if after 4 weeks of Askina® Calgitrol® Ag treatment, there has been no improvement then, in line with accepted wound management practice, the original diagnosis and overall therapy should be reassessed. Askina® Calgitrol® Ag should be left in place as long as possible in order to prevent trauma to the fragile newly formed tissue and to reduce cross contamination through frequent dressing changes. Thick necrosis should be removed before applying Askina® Calgitrol® Ag.  
 Clinicians' Healthcare Professionals should be aware that there is very limited data on prolonged use of the device. The device contains products and particularly in children and neonates. At present there are no contraindications for the pediatric use of the device. Inappropriate treatment, it is recommended that official best practice guidelines for wound management be followed.

**5.5 Russian Safety Standards (GOST-R)**

**7. PRESENTATION**  
 Askina® Calgitrol® Ag is available in the following sizes:  
 • 10 x 10 cm • 15 x 15 cm • 20 x 20 cm  
**Warning:** Re-use of single-use devices creates a potential risk to the patient or user. It may lead to contamination and/or impairment of functional capability. Contamination and/or limited functionality of the device may lead to injury, illness or spread of the patient.

Store in a cool dry place. Away from direct sunlight. 5°C > 25°C. Gamma Sterilized Do not Resterilize. Single use only Batch number. Product code Attention, see instructions for use. Use by: year and month Manufacturer. Do not use if pack is damaged Recyclable.

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