only for US-Users

If you have questions, please contact:

AESCU LAP Inc.
3773 Corporate Parkway
Center Valley, PA 18034
Phone: +1-800-282-9000
www.aesculapusa.com

The *proGAV 2.0* valve was determined to be MR Conditional (ASTM F2503-13).

Non-clinical testing demonstrated that the *proGAV 2.0* valve is MR Conditional.

A patient with this device can be safely scanned immediately after implantation in an MR system meeting the following conditions:

- Static magnetic field of 3 Tesla or less
- Maximum spatial gradient magnetic field of 720-Gauss/cm (7.2 T/m)
- Maximum MR system reported, whole body averaged specific absorption rate (SAR) of 4-W/kg (First Level Controlled Operating Mode)

Under the scan conditions defined above, the *proGAV 2.0* valve is expected to produce a maximum temperature rise of 2.1°C after 15 minutes of continuous scanning.

In non-clinical testing, the image artifact caused by the device extends approximately 20 mm from the *proGAV 2.0* valve when imaged with a gradient echo pulse sequence and a 3-Tesla (3-Tesla/128-MHz, Excite, HDx, Software 14X, M5, General Electric Healthcare, Milwaukee, WI) MRI system.

Patients should register the conditions under which the implant can be scanned safely with the Medic Alert Foundation (www.medicalert.org) or equivalent organization.

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